

INFANT RESUSCITATOR IR 100

EFFECTIVE • INTENSIVE • SLEEK



INFANT RESUSCITATOR IR 100

INFANT RESUSCITATOR SET UP INSTRUCTIONS

Connect Oxygen supply to Gas Inlet Port 4.2 kg/cm²
(or) 5 to 15 LPM

Connect Patient supply tube as to Gas Outlet Port

Turn on Oxygen flow to between 5 to 15 LPM

Adjust Inspiratory Pressure Control to desired Peak
Inspiratory pressure level

Adjust Peep control to the desired level

To resuscitate- Occlude Tee piece with thumb at the
desired rate

Note:

Recommended gas flow rate 5 to 15 LPM
Factory setting of pressure relief valve
40 cm H₂O

SPECIAL FEATURE:

Inbuilt flow meter 0 to 15 LPM
Dual input gas supply
Patient pressure measured near the patient end
Easy to mount any ware

Dimension – 170 (L) x 110 (W) x 180 (H) mm

Weight (approx.) – 2.5kg



Phoenix Medical Systems Private Limited

DP42, SIDCO Industrial Estate, Thirumudivakkam, Tamil Nadu – 600132, India

Toll Free India : 1800 425 3503, Phone: +91 44 2478 1705 / 2227,

✉ India : sales@pmsind.com, International: exports@pmsind.com

🌐 www.phoenixmedicalsystems.com